

Haier

Haier Forehead Thermometer

体温枪

国内定制专用推荐支持

FT- 100B Product features

Haier

Haier infrared forehead thermometer
for medical use

FT-100B



Temperature showing is accurate which is at medical level.

Only one second needed for temperature showing

Long life span especially for continuous using

certificated by CE of EU and FDA of USA


Low voltage warned

FT- 100B Product features

Haier


When the temperature is over 37.3°C, the color will be orange. Sound warning will happen

Fever warning:
between 37.4
°C and 39.1°C

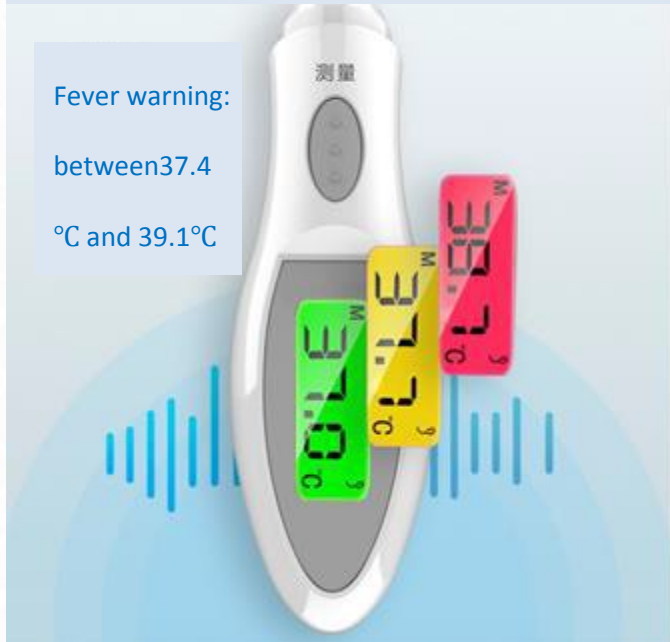


Fever warned,
different
colors for
different
temperatures

Small and
light body for
easily taking
away

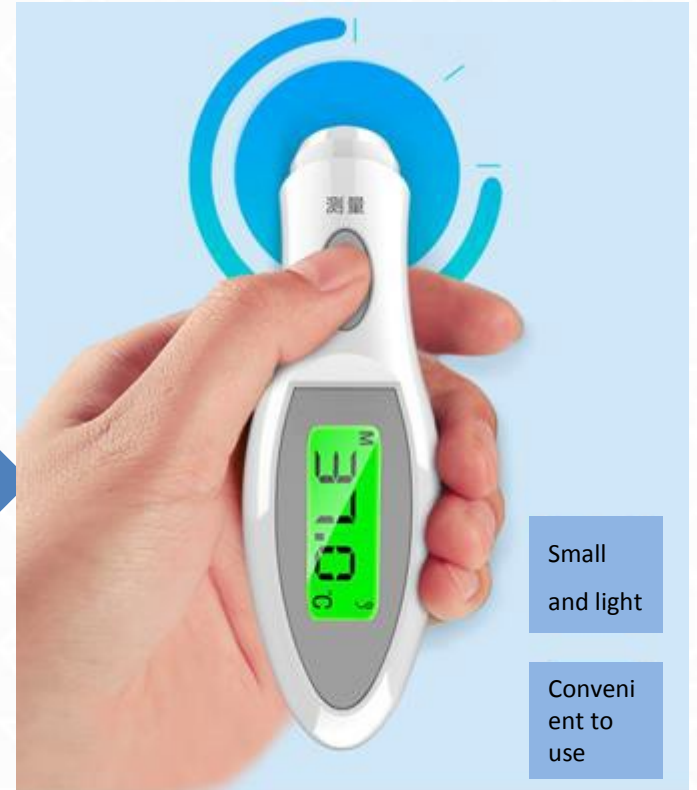

Small
and light


Conveni
ent to
use

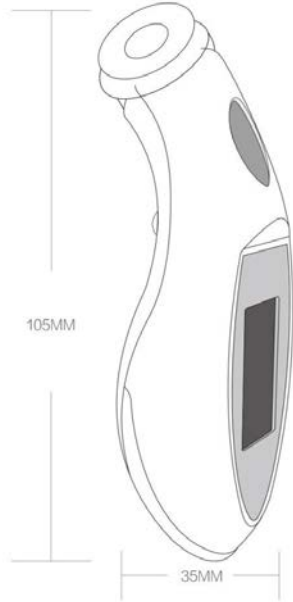

- Measurement method: noncontact
- Measurement range: 32.0°C~42.9°C
- Minimum temperature showing: 0.1°C
- Power consumption: $\leq 30\text{mW}$
- Maximum allowable error: $\pm 0.3^\circ\text{C}$ (ambient temperature 10~40°C)
- Battery type: CR 2032
- Battery life span: 2500 times
- Tips: turned on ,turned off , end of measurement , low voltage etc.
- Ambient temperature: 10°C~40°C, humidity $\leq 85\%RH$



- Press the 'ON/OFF' button to turn on the thermometer.
- The probe shall be aimed at the forehead, from a distance of (1-3)cm, press the 'SCAN' button and then release it.
- Unit switch: with the thermometer off, press the ' 'button for 2 seconds to switch between Celsius and Fahrenheit(As Fig E shows).
- Memory search: press the ' MEM 'button, which will then display the last measured temperature, as Fig F shows, and allows for a view of the last 10 measurements taken.
- The LCD will display as Fig G and cannot display temperature when the ambient temperature is out of the operating ambient.
- Shut off: The thermometer will automatically shut off after of inactivity.

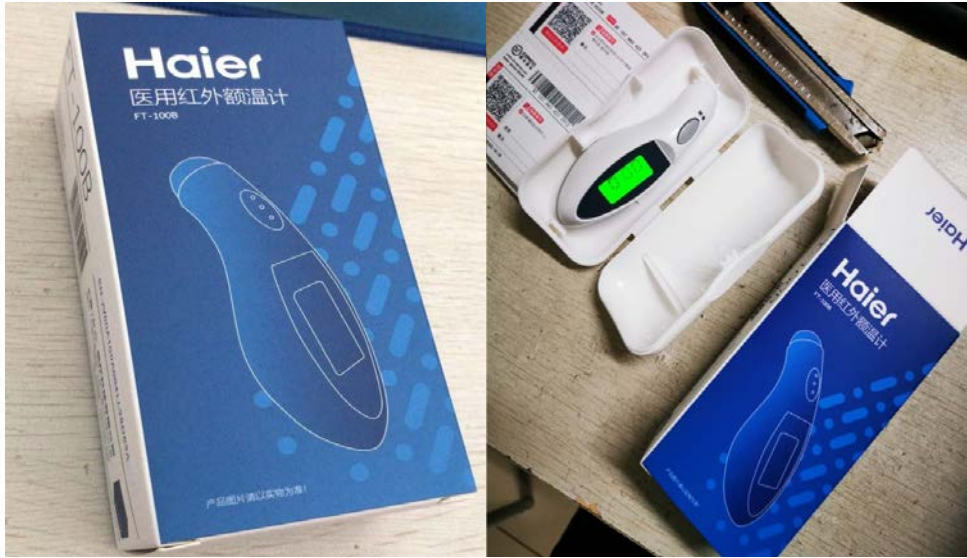

- Dimension: 105mm X 35mm X35mm
- Net weight: 36g
- 100 Sets packaged in 1 big carton, 10.7kg
- Carton: 510mmX296mmX375mm